

# Duplication moulds

---

## Materials & Accessories

### Fabrication:

- Erkoflex, 3.0 - 5.0 mm

### Model preparation:

- High-fusing wax (725 080) for filling bubbles in the plaster

## Hints

- The duplication with Erkoflex is not as precise as the duplication with duplicating compounds. But this very low-cost method is suitable for demonstration, training and planning models.

**1.** Trim the model flat, put it onto the model plate in the thermoforming unit and thermoform.

Allow to completely cool down.



**2.** Put the formed Erkoflex as shown onto a ring or a pot in a way that the negative has contact only at the outside. Put in plaster while form is standing on the vibrating slab.