# ERKOLOC-Pro

DIN EN ISO  
13485: 2003

Erkoform-3/3d  
Erkopress 300 Tp/-ci

1,0 mm, 160 °C  
2,0 mm, 170 °C  
3,0 mm, 165 °C  
4,0 mm, 160 °C  
5,0 mm, 160 °C

1,0 mm, 2:00 min  
2,0 mm, 2:00 min  
3,0 mm, 3:00 min  
4,0 mm, 4:00 min  
5,0 mm, 5:00 min

Erkoform-RVE  
Erkopress 200 E

1,0 mm, 1:00 min  
2,0 mm, 1:55 min  
3,0 mm, 2:40 min  
4,0 mm, 3:25 min  
5,0 mm, 4:30 min

Lagerung • storage • stockage

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# EN ERKOLOC-Pro

## Chemical characteristics:

PETG, hard side (0.6 mm)[1,2,3 + 4 mm]: copolyester  
TPU, soft side (0.4 mm)[1 mm]: polyurethane  
PE, insulating foil: polyethylen  
Harmless to health, BgVV (Federal Health Office)  
listed thermoplasts with tested biocompatibility.  
Insoluble in water, inactive, harmless to ground  
water.

## Disposal/Recycling:

Household waste (combustible, low in pollutants and  
high in energy)/general recycling for plastic.

**Field of use:** (see brochure thermoforming technique  
and material card) All kinds of comfortable occlusal  
and bruxism splints, Silensor-sl. Adjustment is possible  
with the Occluform, by grinding or by addition.

## Material properties:

Durable, resistant double layer plate, hard/soft. Hard  
layer adjustable by addition using acrylics  
(Resilit-S, 817 501 + 817 503). **Attention:** The soft  
layer does not bond to Erkoflex.

**Finishing:** (see brochure thermoforming technique)  
Finishing set Quick 2 (110 877). The insulating foil on  
the soft layer will only be removed after thermoforming  
and finishing. With polish for plastic the hard  
side can be given a high polish.

**Plastification:** (for machines without heating time  
control) The hard side of the Erkoloc-pro plate  
should be heated. Place the foil in the devices in  
such a way that the spacer/insulating foil is pointing  
towards the model. Test softness of foil with an  
instrument. If permanent impressions result, thermo-  
form.

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