ERKOLOC-Pro blu

DIN EN ISO
13485: 2003

Erkoform-3/3d
Erkopress 300 Tp/-ci

2,0 mm, 170 °C
3,0 mm, 165 °C
4,0 mm, 160 °C
5,0 mm, 160 °C

2,0 mm, 2:00 min
3,0 mm, 3:00 min
4,0 mm, 4:00 min
5,0 mm, 5:00 min

Erkoform-RVE
Erkopress 200 E

2,0 mm, 1:55 min
3,0 mm, 2:40 min
4,0 mm, 3:25 min
5,0 mm, 4:30 min

Lagerung • storage • stockage

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EN ERKOLOC-Pro blu

Chemical characteristics:

PETG, hard side (1,2,3 + 4 mm): copolyester
TPU, soft side (1 mm): polyurethane
PE, insulating foil: polyethylen
Harmless to health, BgVV (Federal Health Office)
listed thermoplasts with tested biocompatibility.
Insoluble in water, inactive, harmless to ground
water.

Disposal/Recycling:

Household waste (combustible, low in pollutants and
high in energy)/general recycling for plastic.

Field of use: (see brochure thermoforming technique
and material card) All kinds of comfortable occlusal
and bruxism splints. Adjustment is possible with the
Occluform, by grinding or by addition.

Material properties:

Durable, resistant double layer plate, hard/soft. Hard
layer adjustable by addition using acrylics
(Resilit-S, 817 501 + 817 503). **Attention:** The soft
layer does not bond to Erkoflex.

Finishing: (see brochure thermoforming technique)
Finishing set Quick 2 (110 877). The insulating foil on
the soft layer will only be removed after thermoform-
ing and finishing. With polish for plastic the hard
side can be given a high polish.

Plastification: (for machines without heating time
control) The hard side of the Erkoloc-pro blu plate
should be heated. Place the foil in the devices in
such a way that the spacer/insulating foil is pointing
towards the model. Test softness of foil with an
instrument. If permanent impressions result, thermo-
form.

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