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Nail correction system with precisely adjustable tensile force fits all nail forms for treatment of:

ingrown nails, nails that are involute on one or both sides, hypergranulations, partial resections, resections, irritations in the nail bed groove, etc. ONYCLIP combines simple application with the highest degree of wearing comfort. The correction strips are made of coldhammered special steel (3 mm wide, 0.15 and 0.10 mm thick). The coating used is compatible with the adhesive. The ONYCLIP strip can be fitted to any nail form. Its effective tensile force can be pre-activated as desired. Here you see some examples pictures a - e.

Application:

- Select strip according to nail thickness/nail arch: white 0.10 mm, beige 0.15 mm.
- Measure the width of the nail with the measuring strip (illustr. 1).
- The correct length of the ONYCLIP strip is the width of the nail minus 2 mm.
- Use a diagonal nipper to trim the ONYCLIP strip (illustr. 2).
- Use a grindstone to round off and polish the cut edges (illustr. 3).
- Polish the nail with the grindstone in the adhesion zone on the left and on the right to ensure secure adhesion (illustr. 4).
- If necessary, use round-nose pliers to cut the ONYCLIP strip to match the nail form, keeping in mind the desired elastic force¹ (illustr. 5).
- Degrease nail and ONYCLIP strip (Degreasing agent 613 050).
- Apply a small amount of adhesive gel to one side of the strip (illustr. 6).
- Now press the ONYCLIP strip in place and hold for app. 60 seconds using the spatula (taking care not to slip) (illustr. 7).
- Wait for app. 1 minute, then repeat the procedure on the other side.
- The ONYCLIP strip can be rendered practically invisible after it has dried for 12 hours by applying nail polish. This will not dissolve the adhesive (illustr. 8).

What can go wrong?

ONYCLIP does not adhere properly:


You waited too long to press the strip in place (2 seconds at most). The strip slipped out of position while being held in place. You have pressed too strong. The nail was not degreased thoroughly. The nail surface was not polished smooth enough. The adhesive was too old.

Tensile force too strong for patient:

Carefully remove the ONYCLIP strip by inserting a flat spatula or knife, bend as required and repeat cementing procedure.

¹ The elastic force of the ONYCLIP strips (1 cm ~ 1 g):

Length	spring travel	elastic force (0.10)	elastic force (0.15)
15 mm	2 mm	4 cn	5 cn
20 mm	4 mm	7.2 cn	9 cn
	2 mm	3 cn	4.2 cn
	4 mm	5 cn	7.5 cn

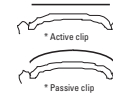Packaging forms:

Package for app. 40 applications. The ONYCLIP strips are 10 cm long and 3 mm wide.

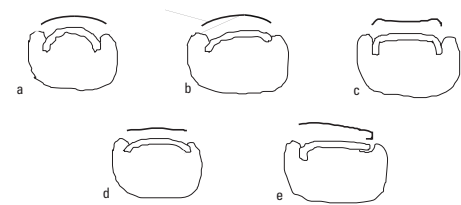
Order no.:

- 270 500** ONYCLIP introductory package, 8 strips, 4 pcs. 0.10 mm thick and 4 pcs. 0.15 mm thick, 1 measuring strip, 1 spatula, 3 g of adhesive gel.
- 270 511** ONYCLIP strips, refill package 0.10 mm thick, 8 pcs.
- 270 516** ONYCLIP strips, refill package 0.15 mm thick, 8 pcs.
- 270 505** 3 g tube of adhesive gel
- 270 502** Diagonal nipper
- 270 503** Round-nose pliers

Instructions for the adhesive gel:

Storage: Store the adhesive gel in a cool dry place when not in use. Storage life: App. 6 months after opening. **Note:** Contains cyanide acrylic ester. Can cement skin and eyelids within sec. Medical aid is necessary in case of contact with eyes or internal organs. Keep out of children's reach.

¹ corresponds to the guide 93/42/EEC for medical devices

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